

STANDARD SHIPPER (CXST1)

LOADING EGGS, SPERM & EMBRYOS INTO CRYOPORT EXPRESS® STANDARD SHIPPER (CXST1)

SECTION A

LOADING SAMPLES FOR SHIPMENT

STEP 1

RELEASING THE RETENTION BAND





Once the white carton is open, you will need to release the retention band on the shipper plug. Pull the small circle handle up to release the locking tab. Next, unlatch and pull apart the straps.

STEP 2



Wearing safety gloves, remove the shipper plug.

STEP 3



Pull up on the black strap to raise the Canister and/or Lifter with gloved hands.


STEP 4A





If the straw/commodity is loaded in a 65mm goblet (Daisy Goblet), place the Goblet onto the lifter and lower the commodity into the dewar.

STEP 4B

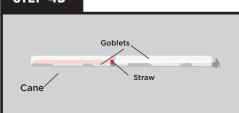

If loading only one goblet on a cane, place a 2nd inverted empty goblet on top to hold the straws in place. If shipping vials, use a cane sleeve over the vials.



Place a cane sleeve over the cane with the goblets or vials.



Ensure the cane sleeve is completely covering the cane, goblets, or vials. WARNING: Straws may come loose if not properly secured.




Safely place the cane with the secured goblets or vials and sleeve into the canister and lower into the dewar.

STEP 5



Replace the shipper plug. Make sure the black strap is left on outside of the shipper plug.

STEP 6

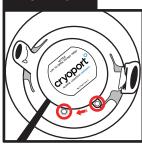



Ensure the hole on the shipper plug and the gray hole on the dewar are aligned. Turn shipper plug to align, if necessary. Also ensure the retention band handles are not overlapping the holes.

IF THE DRY SHIPPER DOES NOT EMIT VAPOR WHEN PLUG IS REMOVED

PLEASE CONTACT CUSTOMER SERVICE IMMEDIATELY, BY CALLING (949) 470-2305 OR EMAIL CS@CRYOPORT.COM



STANDARD SHIPPER (CXST1)

LOADING EGGS, SPERM & EMBRYOS INTO CRYOPORT EXPRESS® STANDARD SHIPPER (CXST1)

SECTION A (continued)


LOADING SAMPLES FOR SHIPMENT

STEP 7

SECURING THE RETENTION BAND




Place the smaller circle handle under the other strap and pull the handle through the slit.

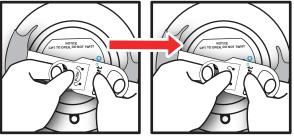

Place your thumb on the word "PUSH" and pull the straps away from each other to place the tab with the word PUSH under the tab with the green check mark.



Ensure the PUSH tab is behind the green check mark tab.

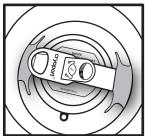

The green check mark will be visible when secured.

STEP 8

If you have non-temperature sensitive accessories you want to include with your shipment, please see the "How to Include and Remove Accessories in the CXST1 Shipper" instructions.

STEP 9





- Close flaps making sure to have flap "A" showing on the outside of the white carton.
- For International shipments: REMOVE AND O SIGN ALL FOUR (4) COPIES OF THE COMMERCIAL INVOICES.
- Place all four (4) signed copies of Commercial
 O Invoices back inside shipping envelope on Flap
 "A" behind Air Waybill.

SECTION B

SHIPPING



Seal top of white carton with packaging tape making sure to have flap "A" showing on the outside of the white carton.



For those shipments containing regulated dangerous goods/hazardous materials, the shipper is responsible for correctly preparing the shipment according to the current International Air Transport Association (IATA) and International Civil Aviation Organization (ICAO) dangerous goods regulations.

This includes correct identification, classification, packaging, dangerous goods markings and labeling as well as completion of all pertinent and required documentation. The shipper is the customer who is preparing the shipment at each leg in the transportation of the dry dewar packaging.

24-hour Help Line (949) 470-2305

Cryoport Systems, LLC • 17305 Daimler St. • Irvine, CA 92614

Phone: 1-949-470-2300 | Fax: 1-949-470-2306

www.cryoport.com • e-mail contact: cs@cryoport.com